

CLEAN FLASH OPA

High Level Medical Instrument and Endoscope Disinfectant

Ready to Use / Liquid

5000 ml



Product Specifications

* CHEMICAL COMPOSITION

Ortho-phthalaldehyde%0,55, Corrosion inhibitors and auxiliary agents.

* PRODUCT SPECIFICATIONS

CLEAN FLASH OPA is ready-to-use and disinfects and sterilize in 5 minutes and usable for maximum 14 days without loss of efficiency. It contains corrosion inhibitor. It can be easily checked with test strips. It is a ready-touse, colourless, odour-free and nontoxic disinfectant for disinfection of medical equipment, flexible and rigid endoscopes and it is phenol-free and formaldehyde-free.

* USAGE

It is suitable for endoscope washing machines and manual use. Fill the solution into the disinfection tank. Disassemble the equipment precleared with enzymatic solution and place in the tank in such a way that all its surfaces will contact with solution, and seal the tank. After keeping it in the tank for the action time, take the equipment out of the solution and rinse with ample water. Efficiency of the solution should be checked with test strips. It is recommended to replace the solution if it gets dirty noticeably. Add glycine of 25-32 g per 5L inactive solution for disposal of it according to the applicable regulations. Please comply with the user instructions given by the manufacturer when you use it with automatic endoscopy washing and disinfection machines.

* INTENDED USE

Clean Flash Opa Solution is very suitable for disinfection of any type of surgical equipment, heatsensitiveanaesthetic accessories (ambu mask, etc.) and all endoscopes such as Karl Storz, Olympus, Pentax, Fujinon, flexible and rigid) and for disinfection of all kinds of surgical equipment and laparoscopy equipment. It does not cause any harm on the endoscope covers such as rust, corrosion, softening, colour change, etc. Clean Flash Opa solution is suitable for disinfection of medical equipment made of rubber, plastic, PVC, acrylic, polymer, polycarbonate, polystyrene, HDPE, porcelain and stainless steel.

* ANTIMICROBIAL CHARACTERISTICS

Bactericidal, Fungicidal, Sporicidal, Virucidal, Tuberculocidal, Mycobactericidal.

Product

Item.-no	Ref:5452
Content	1

Quality Features

ISO / EN	EN13727
ISO / EN	EN14476
ISO / EN	EN13624
ISO / EN	EN14348
ISO / EN	EN13704
ISO / EN	EN1650
CAS	643-79-8
CE	CE 2195, 93/42 EEC

CLEAN FLASH OPA

High Level Medical Instrument and Endoscope Disinfectant


Ready to Use / Liquid

5000 ml



Packing

Quantity Inner Packing	1
Quantity Outer Packing	4
Quantity Per Pallet	40
Product Units Per Pallet	160
Pallet Dimensions cm	120x80
LxWxH Outer Packing in mm	390x270x303
Volume in m ³ Outer Packing	0,03191 m ³
Gross Weight Outer Packing in KG	21,80 KG
Content Wight ml = gr.	5.500 gr.

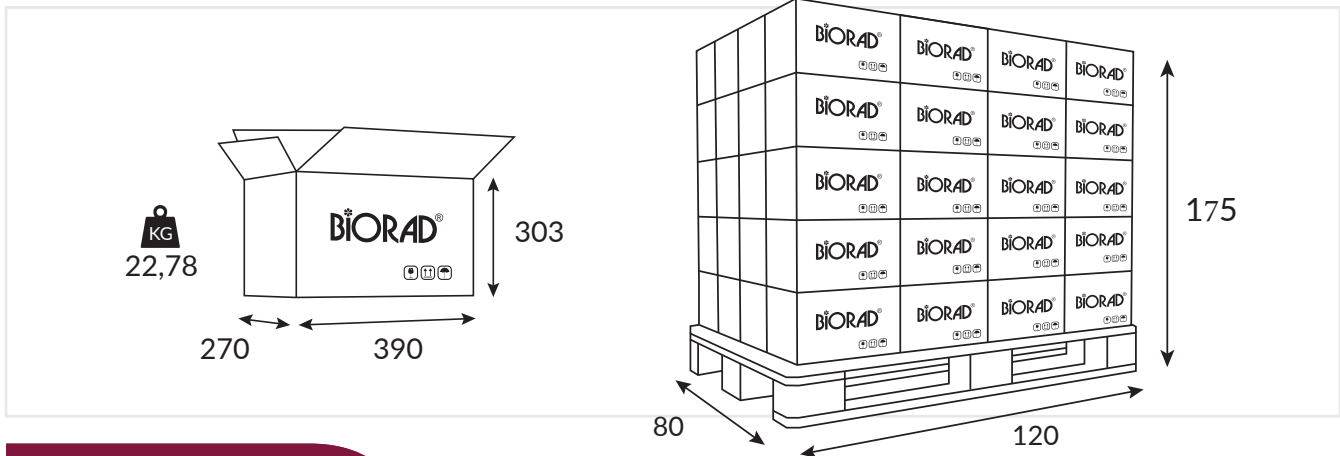

Labeling

HS Code	340130000000
Country of Origin	Turkey
EAN Code Inner Packing	8699856750362
EAN Code Outer Packing	
Shelf Life	3 Years

Do you need more detailed information? We are happy to help!

The technical data are average production values and may vary slightly in individual cases. Subject to changes, no guarantee for printing errors or mistakes.